


# Mepore<sup>®</sup> Ultra

## SHOWERPROOF, SELF-ADHESIVE ABSORBENT DRESSING

Mepore Ultra is a self-adherent, absorbent and breathable dressing with an outer film layer which protects the wound from water and contamination. Its skin-friendly, waterbased, solvent-free adhesive provides a gentle and secure fixation.

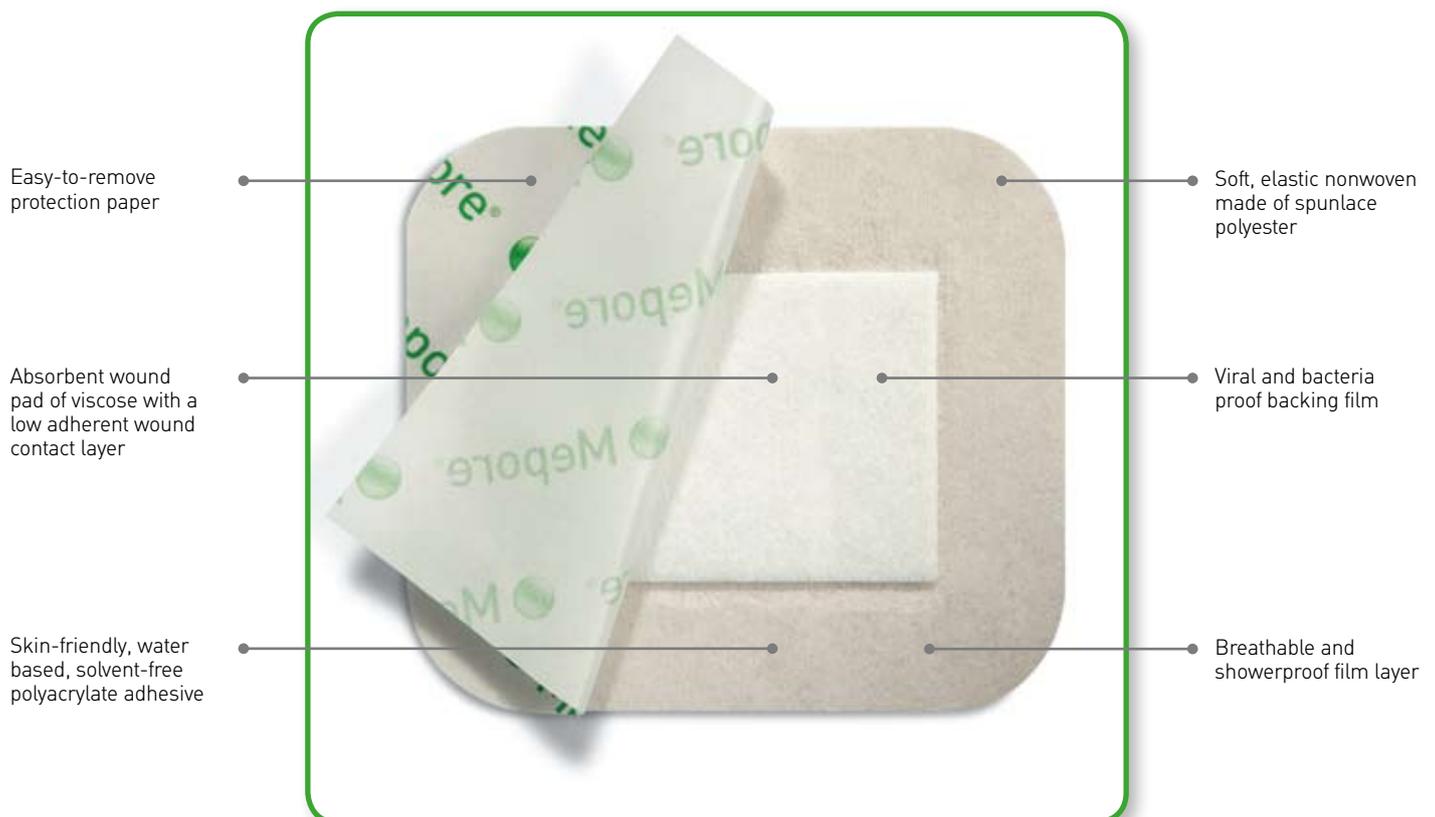

## HOW MEPORE ULTRA WORKS

The backing film allows showering and protects the wound from water and contamination. It also protects clothing and bedlinen from blood or exudate strike-through. Due to a highly absorbent pad and a high vapour permeability of the backing film, the dressing offers good total fluid handling. The polyacrylate adhesive is designed to be gentle to the skin and at the same time provide secure fixation.

The easy-to-remove protection paper permits sterile application, as well as simplifies dressing application. The stability of the fabric/film laminate also supports ease of application.

## HOW TO USE MEPORE ULTRA



1. Open the package and remove the dressing.



2. Grip both overlapping release papers, exposing adhesive surface allowing initial fixation of the dressing.



3. Position the dressing on the skin without stretching and gently remove the rest of the release papers. Firmly smooth adhesive border to obtain proper adhesion. Do not stretch the dressing when applying.

## FREQUENCY OF CHANGE

The frequency of dressing changes will be governed by the condition of the underlying wound. Upon removal grasp the edge and slowly peel the dressing from the skin in the direction of hair growth.

## BENEFITS OF MEPORE ULTRA

- Allows showering
- Low-adherent wound contact layer minimizes risk of adherence to the wound
- Protects wound from water and outside contamination
- Viral and bacteria proof backing film
- Optimized absorption capacity
- Easy-to-remove protection paper allows for convenient and aseptic application
- Breathable film
- Low friction to clothing and bedlinen
- Skin-friendly water-based adhesive
- Rounded corners for secure fixation (all sizes up to 9 x 15 cm)



## AREAS OF USE

Mepore Ultra can be used in a wide variety of wounds which have low to moderate exudate levels. These include surgical wounds, minor burns, cuts, and abrasions.

## PRECAUTIONS

Care should be taken to ensure that Mepore Ultra is not applied under tension, to prevent shearing forces causing damage to the skin. This is particularly important when the dressing is applied over joints.

### MEPORE ULTRA ORDERING INFORMATION

Art. No	Size cm	Pieces Per Inner	PIP Code	NHS Code
680825	7 X 8	60	337-7041	EIJ008
680925	10 X 11	36	337-7033	EIJ062
681025	11 X 15	36	337-7058	EIJ063
671125	9 X 20	24	297-9821	EIJ029
671225	9 X 25	24	272-3583	EIJ011
671325	9 X 30	24	297-9839	EIJ030



Mölnlycke Health Care Ltd, Arenson Centre, Arenson Way, Dunstable, Bedfordshire LU5 5UL, UK. Tel. +44 (0) 870 6060766, [www.molnlycke.co.uk](http://www.molnlycke.co.uk)

The Mölnlycke Health Care name and logo, Mepore® Ultra are registered trademarks of Mölnlycke Health Care AB. ©2009 Mölnlycke Health Care. All rights reserved.