

## KaWe PERSON-CHECK®





Dear Customer, thank you for choosing a KaWe product. Our products are known for their high quality and long life.



**Please read this User's Manual thoroughly and carefully before attempting to use this product and heed the care instructions.**

**Utilization:** The KaWe PERSON-CHECK® measuring instruments are only to be used by authorized personnel. The KaWe PERSON-CHECK® instruments are intended for the measurement of the human body size.

**Use:** The instruments are used for the simple measurement of the human body size by placing the movable jaw blade on the head of the person to be measured. The measured value (in cm) can be read from the tape.

**Unsuited use/contraindication of the product:**

Any use other than that described here is not in accordance with the intended use of the unit. The manufacturer is not liable for any resulting damages. There are no known risks associated with the use of this product. The user alone bears the risk.

**Further information, maintenance, storage:** When used and stored properly, the KaWe PERSON-CHECK® measuring instrument will serve you many years.

**Guarantee:** When used properly and in accordance with the User's Manual we guarantee this product for two years from the date of purchase. Should you need further information or should your instrument require repair, please contact your dealer.



**Warning instructions:** Do not pull the measuring tape out past its maximum measuring length of 2,10 meters.

**Limitation of reuse preparation:** Frequent reuse only minorly influences this product. The working life of the instrument usually depends on the wear and tear and damage by use.

**Instructions**

**Storage and transport:** No special requirements.

**Cleaning preparation:** No special requirements.

**Manual cleaning:** KaWe PERSON-CHECK® can be cleaned exteriorly with a damp, soft and lint-free cloth.

**Automatic cleaning:** Automatic cleaning is not possible/necessary.

**Disinfection:** Disinfection of the measuring arm. Disinfecting and cleaning solutions such as Meliseptol®/Meliseptol® rapid are to be used in accordance

with manufacturer instructions. Please heed the given reaction times! Never use abrasive cleaning agents!

**Maintenance:** Defective parts must be removed and/or replaced.

**Check and performance test:** Check all movable parts such as jaw blades. Perform visual inspection for damage and wear.

**Packaging:** No special requirements.

**Sterilization:** Sterilization is not possible/necessary.

**Storage:** No special requirements.







**Common accessory parts:** Further information about this article can be obtained by visiting our website at: [www.kawemed.de](http://www.kawemed.de).


Manufacturer: **KaWe**

**Contacting the manufacturer:** Address or phone number of the dealer or dial +49-7141-68188-0.

**Disposal:** This product is to be disposed of at a separate collection site in accordance with local regulations.

#### Symbol key:

	Manufacturer
	Date of manufacture
	Lot code
	Heed the User's Manual
	Caution!
	Complies with relevant EU guidelines

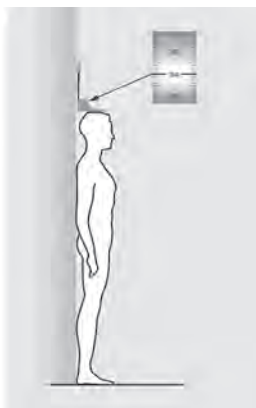



- de** Alle Angaben ohne Gewähr – Änderungen vorbehalten.
- en** All information is without guarantee and subject to change.
- fr** Informations sous toutes réserves – Sous réserve de modifications.
- it** Tutte le informazioni sono fornite senza alcuna garanzia e possono essere modificate.
- es** Toda la información sin compromiso. Nos reservamos el derecho de realizar cambios.
- pt** Todas as indicações entendem-se sem compromisso – Sujeito a alterações sem aviso prévio.
- ru** Информация представлена без гарантий любого рода – оставляем за собой право вносить изменения.



**KIRCHNER & WILHELM** GmbH + Co. KG  
Eberhardstr. 56 • 71679 Asperg • Germany

**Zentrale / Central office**

Fon: +49 -7141-68188-0

Fax: +49 -7141-68188-11

Email: [info@kawemed.de](mailto:info@kawemed.de)

Internet: [www.kawemed.de](http://www.kawemed.de)